

GEMINI CAUTERY KIT Part No. RS-300



B&B Medical Instruments Co., Inc.
Cathetersburg, MD 20611
Ph: 888-424-2988 Fax: 888-424-8111

The GEMINI CAUTERY SYSTEM

is the perfect tool for your cauterizing needs. The unique tip sizes combined with two different operating temperatures allow for maximum versatility. The convenient battery spacer converts the hand piece from high temperature to low temperature.

TECHNICAL DATA:

Length (without tip): 140.7mm
Weight (with two AA batteries): 70g
Electrical: 2 volts, 1.0 volts
Tip Sizes: 3x 1mm, 50x 1mm

The Gemini Cautery Unit has the following features:

- Simple tip design for easy replacement
- Uses standard AA batteries
- Easy screw cap for convenient battery replacement
- Hard plastic case with die cut foam for convenient storage
- Battery spacer for optional temperature operation
- One year warranty

Tip Replacement



1. Grasp tip by the plastic body to avoid damage to the cutting wire.
2. Line up the tip to go into the tapered end of the tube.
3. Slide tip in until the plastic body and hand piece meet.
4. To remove the tip, grasp and pull tip free.

Replacement 5mm Tip Box of 10
Part No. RS-301

Replacement 10mm Tip Box of 10
Part No. RS-302

Battery Replacement



1. Unscrew the end cap by turning it counter-clockwise.
2. Insert your new AA batteries into the tube, positive end first.
3. After both batteries are in, screw the end cap back on.
4. To remove the batteries, remove end cap, place hand over opening, tilt the cauterizing unit up to allow the batteries to slide free of the hand piece. If batteries do not slide out, a gentle shake may be required.

Low Temperature Operation

Included with your cauterizing unit is a battery spacer enabling you to use your cauterizing unit at a lower temperature setting. To use this lower setting, simply remove the second battery and replace it with the multi-directional battery spacer. Your cauterizing unit now works at half the temperature. Average temperature with two batteries 2200°F/1204°C. (1300°F/704°C with battery spacer)

GEMINI CAUTERY KIT Part No. RS-300

WARNING: DO NOT USE ON HUMANS

Do not use in the presence of flammable materials.

Always use Alkaline batteries.

Do not incinerate.

CONTACT YOUR DEALER FOR MORE INFORMATION